



ETT | TTFix

HSO[®]
HOSPITAL SUPPLY ORGANISATION



ENDOTRACHEAL AND
TRACHEOSTOMY TUBE
HOLDER SYSTEMS

**BREATH-
ING FREE**

Two challenges – three solutions

The secure attachment of tracheostomy and endotracheal tubes is of vital importance for the patient and allows doctors and the nursing staff to provide the optimal care.

The tube holder systems prevent the accidental extubation or slippage of tracheostomy and endotracheal tubes. The single-piece ETTFix for endotracheal intubation is available either with Velcro fastener or in a silicone version. The two-part TTFix is used to secure tracheostomy tubes.

ETTFix One ENDOTRACHEAL TUBE HOLDER



ETTFix One is a single-piece strap system for the **secure attachment of endotracheal tubes**. Thanks to its skin-friendly material, the strap is especially suited for patients with skin irritations, burn injuries or beards. A Velcro fastener prevents the tube from slipping and minimises the risk of accidental extubation.

ETTFix One can be used in **two different ways**: starting with the strap at the tube or at the back of the head. Both alternatives are equally safe. Please note, however, that the Velcro strip must always be attached directly to the tube!

Order options

Product	Shipping unit	Order No.
Single, white	25 pieces	HSOETT2000

ETTFix Silicone ENDOTRACHEAL TUBE HOLDER



ETTFix One Silicone represents a further development of the basic product ETTFix One. Both products allow the secure attachment of endotracheal tubes. The silicone coating along the elastic strap ensuring a firm grip of the tube is an important improvement of ETTFix One Silicone.

The holder literally clings to the tube, providing a secure hold even with patients who salivate heavily. The product does not need to be glued in place, thus eliminating the problem of unwanted adhesive residues. Important to note is that the endotracheal tube must be dry when ETTFix One Silicone is applied.

Order options

Product	Shipping unit	Order No.
Single, blue	25 pieces	HSOETT2010

TTFix TRACHEOSTOMY TUBE HOLDER



TTFix Two is a two-part strip with Velcro fastening system to quickly and easily secure tracheostomy tubes. The flexible strap can be used on patients with different neck sizes. TTFix Two is compatible with all the usual cannula shields. The strap should only be shortened on the side on which there is no Velcro fastener.

Order options

Product	Shipping unit	Order No.
Pair, white	25 pieces	HSOTTFIX3010

www.hso-solutions.at

If you have any questions, please contact
your personal product specialist:



SALES

Rudolf Heintel Medizintechnik GmbH
Erdbergstraße 166
1030 Vienna, Austria

T +43 (1) 403 89 56-0

F +43 (1) 403 89 56-50

medizintechnik@heintel.at

www.heintel.at



MANUFACTURER

MSP Medizintechnik GmbH
Würtzlerstraße 16
1030 Vienna, Austria

T +43 (1) 403 89 56-0

F +43 (1) 403 89 56-50

info@hso-solutions.at

www.hso-solutions.at

www.thoraxsafe.com

Photo:

© IStock Photo Arand

© IStock Photo Anton Novik

Subject to errors and omissions
Version 1.0 | Last updated: Sep 2017