

# Blue Ankle Distractor

## Non-invasive Distraction System



### Product properties

The innovative design of the Blue Ankle Distractor made by HSO allows for a quick and individual distraction for the arthroscopy of the ankle (articulatio talocruralis).

The leading surgeon carrying out the arthroscopy is directly connected with the Blue Ankle Distractor via hipbelt and can individually and directly determine the tensile force on the upper ankle joint of the patient.

The distraction of the ankle joint is therefore effected through a focused weight shift by the surgeon. The non-invasive distraction system not only facilitates intraoperative control but also enables a better focused individual distraction force.

The components of the distraction system are of high quality and offer both the patient and the surgeon a secure and comfortable hold.



Place the Blue Ankle Distractor over the joint of the patient. Make sure that the "Front" band lies on the instep. The label "Front" must be readable. The "Under Feet" band must lie below the arch of the foot and the "Back" band over the heel, on the Achilles tendon.



Adjust the Blue Ankle Distractor to the joint of the patient and additionally stabilize the fixation over the "Under Feet" band. This prevents slipping when no tractive force is applied. Put on the hip belt and easily and comfortably run the fastening hooks through the lateral loops of the Blue Ankle Distractor.

From January 2021 only one model:

## ORDER OPTIONS HSO BLUE ANKLE DISTRACTOR

Product	Shipping unit	Order no.
Universal	1 package (10 Sets - single packed)	HSOSAAD-U

CE 0408